

# 1 Safety



✓ Call for help

**EMERGENCY  
PHONE 000  
or 112 on** 

# 2 Response



Wake up Baby: Touch & Shout

Responds

✓ Reassure, make comfortable

No Response

# 3 Airway



✓ Check airway  
✓ Hold head in neutral position. No head tilt

✓ Clear airway



If required, support & turn over to clear airway



# 4 Infant Breathing

Look, Listen & Feel for Breathing

Breathing

Not Breathing Normally

✓ Make comfortable  
Reassure

✓ Observe:  
Airway,  
Breathing  
& Circulation



✓ Hold head in neutral position. No head tilt  
✓ Seal baby's mouth & nose  
✓ Give 2 puffs of air



# 5 Infant Compressions

## Infant Compression

Using 2 fingers, compress 1-2cm on the lower half of the breast bone.

- ✓ 30 compressions & 2 breaths
- ✓ Repeat until ambulance arrives



Compressing chest



# Trouble Shooting Chart

Symptom	Possible Cause	Solution
Alarm activates	Breathing Stopped for more than 20 Seconds	<ul style="list-style-type: none"> <li>Follow the directions on Resuscitation Chart</li> <li>Call Emergency Services</li> <li>Call us</li> </ul>
	Breathing dropped below 10 breath per minute	<ul style="list-style-type: none"> <li>Follow the directions on Resuscitation Chart</li> <li>Wake up the baby</li> </ul>
	Two sensors are connected but the baby does not crawl yet	<ul style="list-style-type: none"> <li>Connect only one sensor</li> </ul>
	One sensor is connected and the baby crawled to the other side of the bed	<ul style="list-style-type: none"> <li>Connect two sensors</li> </ul>
	Bottom of the bed is not rigid	<ul style="list-style-type: none"> <li>Place a rigid bottom such as a plywood board</li> </ul>
	Incorrect sensor placement	<ul style="list-style-type: none"> <li>Place the sensors as instructed in the guide</li> </ul>
	Sensor not connected to control unit	<ul style="list-style-type: none"> <li>Place the sensors as instructed in the guide</li> </ul>
Very long delay until alarm activates	30 seconds seems along time while waiting	<ul style="list-style-type: none"> <li>Please watch the time</li> </ul>
	Environmental factors detected by the monitor such as fan, air conditioning, electrical appliances toy mobile vibrations	<ul style="list-style-type: none"> <li>Find and remove environmental factors.</li> <li>Conduct test as per instructions.</li> </ul>
No response when turning on monitor	Missing or misplaced batteries	<ul style="list-style-type: none"> <li>Check that there are <b>4</b> 'AA' batteries correctly installed</li> </ul>
Low battery warning light or constant clicks	Batteries running out	<ul style="list-style-type: none"> <li>Replace batteries</li> </ul>

Please call **1800 81 81 81** if you are having any technical difficulties